



Instructions for use: The Vapocid®-Tray-System

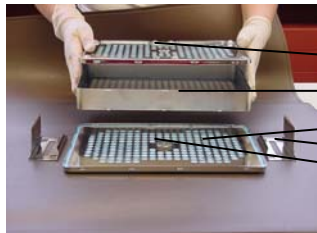

Vapocid®-Tray System stainless steel

General view

- 1 Cover top/bottom
- 2 Stainless steel container
- 3 Filter holder with filter
- 4 Slide lock
- 5 Sealing clip (stainl. steel)
- 6 Nut

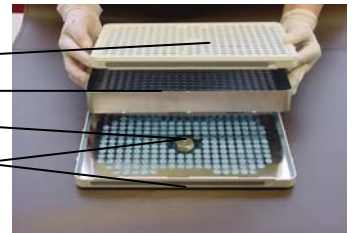

Vapocid®-Tray System plastic

The Vapocid®-Tray System can be used as a working tray, a washing tray and also a sterilising tray for keeping instruments sterile.

Washing procedure: The top and bottom covers must NOT remain on the Vapocid® -Tray during the washing procedure (if necessary, use wash tray cover, *available separately*). At the end of the disinfection procedure, the instruments must be checked to ensure that they are clean.

Sterilisation procedure: Fit the covers (top/bottom), and seal with the sealing clips / slide locks.

Working procedure: Remove both covers just before the treating doctor starts to use the tray (danger of contamination!).

Assembling/dismantling the Vapocid®-Tray:

Place the container in the bottom cover (a) and put the top cover on.



a

Plastic cover

Press the top cover against the bottom cover and slide the slide locks in the direction marked „close / open“ (b)



b

stainless steel cover

Press the top cover against the bottom cover and place a sealing clip on each of the narrow sides. (c)



c

Fitting of the instrument holder (d)



d

Dismantling the filter: Undo nut, remove the filter holder and filter.



e

Fitting the filter: Place the filter with the seal facing downwards (e) and use your fingers to press the seal into the recess on the cover (f). Position the filter holder on the hexagon screw and tighten nut manually (recess on nut facing downwards) (g).



f



g



Important: The filter elements should be changed at least every 3 months and they must be changed immediately if damaged.

Maintenance: Clean the marking space on the sealing clips from time to time.